

SKLAR CHEEK RETRACTORS

RECOMMENDED CARE & CLEANING INSTRUCTIONS

CAUTION: The following instructions are for the Sklar Cheek Retractors. Read instructions prior to use.

GENERAL INSTRUCTIONS

- General Sklar Cheek Retractors are latex free and require no special handling.

- Cleaning Immediately after use, thoroughly rinse fluids / debris from the device with copious amounts of cool running water. Clean with soap and water. Visually check to ensure surfaces are clean and free from damage, stains and bioburden. Rinse Retractors with distilled or de-mineralized water if possible. Inspect device for visual damage, wear and tear or deterioration. Discard broken, defective or worn-out products.

NOTE: When used as directed, Sklar Enzymatic™ (10-2777) and Sklar Kleen™ (10-2701, 10-1630, 10-1613) are ideal cleaning agents and safe for use on plastics.

- Sterilization If steam sterilization is desired, autoclave @ 120°C / 248°F for 30 minutes. Always follow your autoclave manufacturer instructions and adhere to AAMI ST79 standards for safe practices.

NOTE: It is the responsibility of the reprocessor to ensure that the reprocessing is actually performed using equipment, materials and personnel in the reprocessing facility to achieve the desired result. This requires validation and routine monitoring of the process. Likewise any deviation by the user from the instructions provided must be properly evaluated for effectiveness and potential adverse consequences.

P/N	Description	Size
75-1140	Cheek Retractor, Clear Plastic, 2/Pk	Adult
75-1141	Cheek Retractor, Clear Plastic, 2/Pk	Child
75-1143	Cheek Retractor with Tab, Clear Plastic, 2/Pk	Adult

