

SKLAR DECANNULATION STOPPERS RECOMMENDED CARE & CLEANING INSTRUCTIONS

CAUTION: The following instructions are for all Sklar Decannulation Stoppers. Read instructions prior to use. Always process new devices prior to initial use.

GENERAL INSTRUCTIONS

| | |
|---------------|---|
| Handling | The Decannulation Stopper with attached string should be cleaned thoroughly prior to initial use. Handle with care during transportation, cleaning, and storage. |
| Inspection | The Decannulation Stopper with attached string must be inspected prior to each use. The string should be carefully inspected for its connection to the stopper and for any fraying or breakage. The Decannulation Stopper should also be examined for any breakage or wear. Damaged or worn parts should never be used. |
| Cleaning | The entire Decannulation Stopper with attached string can be cleaned manually with soapy water. Sklar Kleen™ Powder (10-1607) and Sklar Kleen™ Low Foam (10-2701) formulas are effective manual cleaning agents. Rinse thoroughly with water (use distilled or demineralized water if possible) and dry (air dry or use a lint-free cloth) before inserting into the Tracheal Tube. |
| Sterilization | CAUTION: Autoclaving, dry heat and cold-soak sterilization are not recommended. |

SKLAR DECANNULATION STOPPERS

| P/N | Description |
|---------|--------------|
| 78-3130 | Full Size #0 |
| 78-3131 | Full Size #1 |
| 78-3132 | Full Size #2 |
| 78-3133 | Full Size #3 |
| 78-3134 | Full Size #4 |
| 78-3135 | Full Size #5 |
| 78-3136 | Full Size #6 |
| 78-3137 | Full Size #7 |
| 78-3138 | Full Size #8 |
| 78-3139 | Full Size #9 |

