

SKLAR DECANNULATION STOPPERS RECOMMENDED CARE & CLEANING INSTRUCTIONS

CAUTION: The following instructions are for all Sklar Decannulation Stoppers. Read instructions prior to use. Always process new devices prior to initial use.

GENERAL INSTRUCTIONS

Handling	The Decannulation Stopper with attached string should be cleaned thoroughly prior to initial use. Handle with care during transportation, cleaning, and storage.
Inspection	The Decannulation Stopper with attached string must be inspected prior to each use. The string should be carefully inspected for its connection to the stopper and for any fraying or breakage. The Decannulation Stopper should also be examined for any breakage or wearing. Damaged parts should never be used.
Cleaning	The entire Decannulation Stopper with attached string can be cleaned manually with soapy water. Sklar Kleen™ Liquid (10-1627), Sklar Kleen™ Powder (10-1607) and Sklar Kleen™ Low Foam (10-2701) formulas, as well as Sklar Enzymatic™ (10-2776) are effective manual cleaning agents. Rinse thoroughly with water (use distilled or demineralized water if possible) and dry (air dry or use a lint-free cloth) before inserting into the Tracheal Tube.
Sterilization	CAUTION: Autoclaving, dry heat and cold-soak sterilization are not recommended.

SKLAR DECANNULATION STOPPERS

P/N	Description
78-3130	Full Size #0
78-3131	Full Size #1
78-3132	Full Size #2
78-3133	Full Size #3
78-3134	Full Size #4
78-3135	Full Size #5
78-3136	Full Size #6
78-3137	Full Size #7
78-3138	Full Size #8
78-3139	Full Size #9

