

SKLAR PESSARY & OS FINDER DEVICES RECOMMENDED CARE & CLEANING INSTRUCTIONS

CAUTION: The following Physician instructions are for all Sklar Pessary & OS Finder Devices. Read instructions prior to use.

GENERAL INSTRUCTIONS

Handling	Devices must be cleaned thoroughly prior to initial use. Handle with care during transportation, cleaning and storage.
Cleaning	The Pessary and OS Finder devices must be inspected prior to each use. To ensure proper cleaning, submerge Pessary or OS Finder in an enzymatic cleaner for 20 minutes. We recommend Sklar Enzymatic™ (10-2776) as an effective cleaning agent. Visibly check devices to ensure surfaces are clean and free from damage, stains and debris. Rinse thoroughly and allow to dry. Follow with a high level disinfectant such as Sklar-Cide™ (10-1623) and rinse thoroughly. Always follow manufacturer's instructions. Discard damaged or defective devices. Store in a clean, dry environment.
Sterilization	<ol style="list-style-type: none"> Process devices individually or in sets. Autoclave devices according to AAMI ST79 standards. Sklar OS Finder and Ring, Gelhorn and Cube Pessaries have been validated for the following steam sterilization cycles: <ul style="list-style-type: none"> - Pre-vacuum Steam, 4 min. at 132°C - 134°C/270°F - 273°F. - Gravity Displacement Steam, 30 min. at 121°C/250°F.

CAUTION: Donut Pessaries cannot be sterilized in an autoclave.

OS FINDER

96-4150	Sklar OS Dilator Set	Teflon reusable set includes OS Locator, Canal Finder, Fundus Dilator.
---------	----------------------	--

PESSARIES

Ring with Support		Gelhorn with Drain		Donut		Cube with Drain	
90-5291	#1	90-5282	#2	90-5271	#1	90-5301	#1
90-5392	#2	90-5283	#3	90-5272	#2	90-5302	#2
90-5393	#3	90-5284	#4	90-5273	#3	90-5303	#3
90-5294	#4	90-5285	#5	90-5274	#4	90-5304	#4
90-5295	#5	90-5286	#6	90-5275	#5	90-5305	#5
90-5296	#6			90-5276	#6		
90-5397	#7						