


NEW!



High Level Disinfecting/Sterilizing Solution

Sklar-Cide™

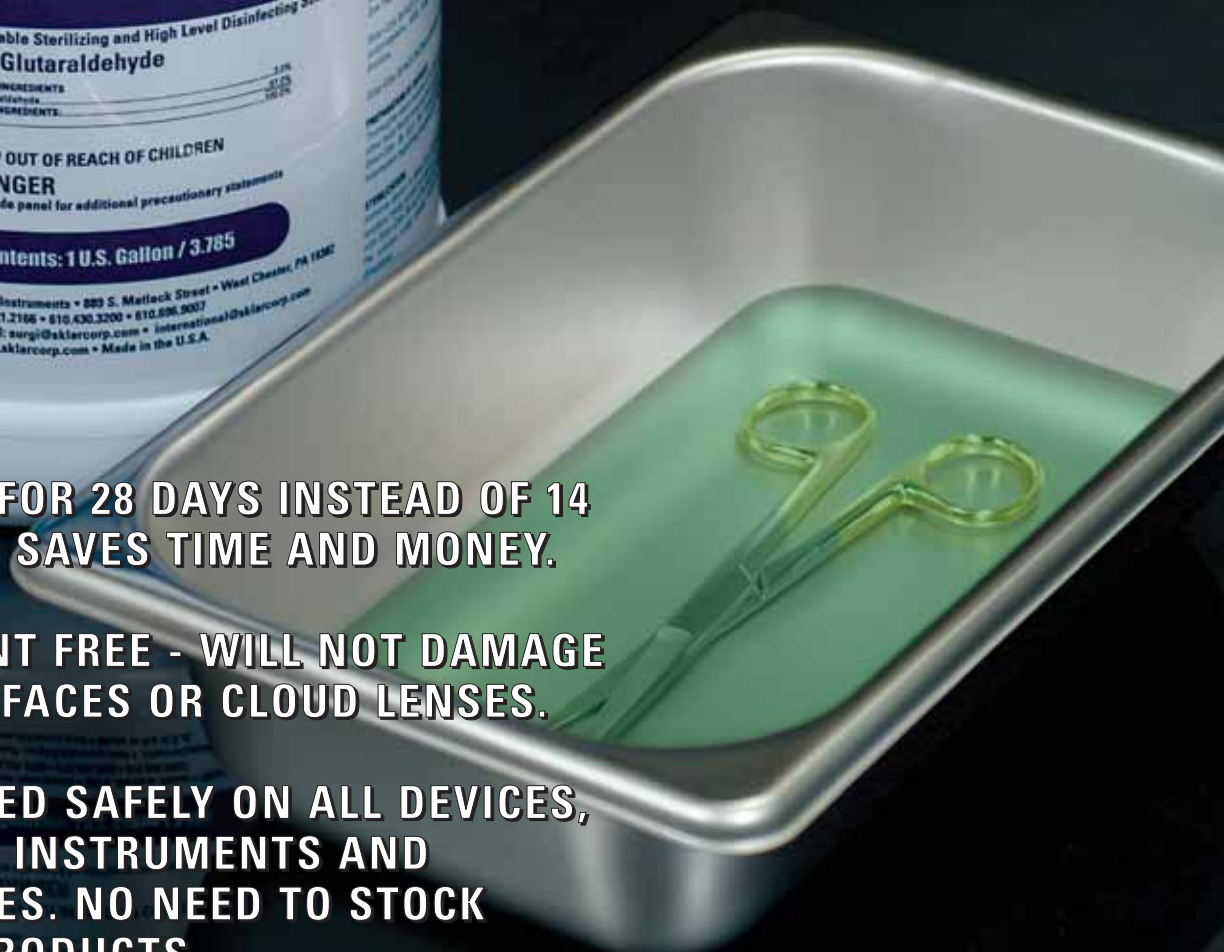

- **REUSABLE FOR 28 DAYS INSTEAD OF 14 SAFER AND SAVES TIME AND MONEY.**
- **SURFACTANT FREE - WILL NOT DAMAGE SCOPE SURFACES OR CLOUD LENSES.**
- **CAN BE USED SAFELY ON ALL DEVICES, INCLUDING INSTRUMENTS AND ENDOSCOPES. NO NEED TO STOCK SEVERAL PRODUCTS.**
- **SPECIAL 10 DAY PRICING AT WWW.SKLARCORP.COM**

www.sklarcorp.com

SKLAR-CIDE²⁸ HLD™ *HIGH LEVEL DISINFECTING SOLUTION*

28-day reusable Sterilizing and High-Level Disinfectant Solution for Medical, Surgical, Endoscopic and Dental Instrumentation



Sporicidal • Bactericidal Virucidal • Fungicidal

Sklar-Cide™ provides a safe and efficient means for the high-level disinfection or sterilization of medical, ophthalmic, dental and surgical devices including endoscopes and lensed instruments. It may be used as a standard soak solution or in automatic reprocessing units set at room temperature. Unlike other solutions, there is no need to increase the solution temperature to provide high-level disinfection in 25 minutes.

Sklar-Cide™ is a surfactant-free, activated glutaraldehyde liquid chemical sterilant and high-level disinfectant. Its active ingredient is glutaraldehyde in a nominal concentration of 3%. When the powered activator is added, the solution turns green in color to indicate its readiness for use. In addition to the active ingredient, this ready-to-use solution contains buffers, wetting agents, and a rust and corrosion inhibitor and it has a pleasant fragrance.

Sklar-Cide™ produces high-level disinfection at room temperature in as little time as 25 minutes, and complete sterilization in 10 hours at room temperature.

SKLAR-CIDE²⁸ HLD™

Item# 10-1623 Case of 4 Gallons

**Item# 96-4105 Glutaraldehyde Strip,
2.1% MEC**

**FOR PRICE QUOTES OR SAMPLES, CONTACT YOUR SKLAR ACCOUNT
MANAGER OR YOUR DISTRIBUTOR SALES REPRESENTATIVE**

USA SALES

Phone: 610.430.3200

800.221.2166

Fax: 610.696.9007

INTERNATIONAL SALES

Phone: 610.430.3200

Fax: 610.430.3941

international@sklarcorp.com